Ladysoft®

Gleitgel

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Ladysoft® Lubricant

About one third of all women suffer from insufficient formation of moisture at the vaginal entrance which may be one cause for pain during sexual intercourse. Experienced gynecologists therefore recommend Ladysoft® as moisturizing gel in order to compensate for insufficient lubrication. This is a natural way for you to experience unimpaired sexual intercourse. The use of condoms also becomes easier and more comfortable. Ladysoft® has an excellent lubricating effect, is fat-free and optimally adapted to the application area.

Special Properties of Ladysoft®

Ladysoft® can be safely used over a long period of time. It does not interfere with your hormonal balance and is particularly skin-friendly. It has no contraceptive effect. Other fat-containing products, such as skin creams, lotions, etc., should by no means be used as lubricants. Contrary to Ladysoft® these products may stick to the mucous membranes and cause irritations (of the skin). Ladysoft® does not attack the sensitive material of condoms. Ladysoft® is absolutely neutral, fat-free, and leaves no stains.

Application

Before intercourse apply a thin layer of Ladysoft® to the vaginal entrance with your fingertips. When using a condom, a slight wetting of its tip is sufficient. One to two drops will suffice in most cases. When using pessaries, Femcones, or Fembowls in order to strengthen the pelvic floor muscles, Ladysoft® facilitates introduction of the cones or pessaries. This will substantially improve training results.

Note

Ladysoft® should not be used if patients complain of pain or redness in the vestibule of the vagina or in the labia area. In these cases a gynecologist should be consulted. Side effects have so far not been reported.

Usable until

LOT

Date of construction

Producer

Medical device

Item number (MED...)

Attention

Read and follow the instructions for use

Single patient – multiple use

CE-mark with identification number of the notified body

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