Cerclage-Pessar

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CE₀₄₈₃

Made in Germany

M E D E S I G N
Dietramszell · Dubai · München

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	Deutsch	4
	English	9
	Français	15
	Italiano	21
	Español	27
	Türkçe	33
	Polski	39
	Русский	45
	中文	51
	يبرع	57

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A woman's profile is shown in silhouette against a blue background. In the upper right corner, a large, vibrant yellow sunflower is partially visible. The overall aesthetic is clean and professional.

Cerclage Pessary

 Instructions for use

When is therapy with a Cerclage Pessary useful?

Cerclage Pessaries are usually used to remove pressure from the uterus during pregnancy. This can contribute to the prevention of a possible premature delivery. Cerclage Pessaries are used in the case of a shortened or open cervix. The use of a Cerclage Pessary is further recommended in the following cases:

- You have already had a premature delivery, which was not caused by an infection.
- You have had a multiple pregnancy.
- You are exposed to increased physical strain (e.g. standing for a long time) during pregnancy.
- To test the activity of your pelvic floor.
- A severe descensus of the uterus has been diagnosed (painful downward pressure caused by physical strain).

What does the Cerclage Pessary do?

The Cerclage Pessary supports the cervix far up and/or turns it toward the sacrum without closing it. This leads to ideal pressure relief.

medesign's Cerclage Pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. The pessaries are very flexible and can thus be folded and inserted without pain (optimally by using a lubricant such as Ladysoft). You will not be able to feel the pessary if it is placed correctly.

How does treatment with the Cerclage Pessary work?

Treatment with the Cerclage Pessary should begin approximately between the 15th and the 20th week of pregnancy. Selecting the correct size is a prerequisite for ideal treatment and freedom of symptoms. medesign's Cerclage Pessaries are offered in various sizes. Your gynaecologist will determine the right size for you.

Your gynaecologist will examine whether the internal os is open or whether the entire cervix is shortened, by means of a vaginal ultrasonography. Further, you will be examined for an infection.

The right sized pessary will then be inserted with the curvature facing upwards. This way the large diameter supports the cervix by being placed on the sacrum and/or the muscles of the pelvic floor. The pessary is placed so that the cervix rests in the upper diameter of the ring. Slight swelling (oedema) of the cervix may be a desired effect. Possible descensus complaints are directly remedied after insertion. If the Cerclage Pessary is inserted with the curvature facing downwards, ideal support cannot be achieved.

Further examinations will allow your gynaecologist to regularly check that the pessary does not cup the cervix too tightly. Should you be free of complaints, regarding descensus, after the pessary was inserted, it may remain in its place until delivery. If the pessary was inserted in order to avoid a possible miscarriage, it is removed in the 37th week of pregnancy. The pessary may only be checked and changed by healthcare professionals.

During treatment with the Cerclage Pessary, your gynaecologist will regularly (every or every other week) check the pH-value of your vagina and at the same time clean and disinfect the inserted pessary. To support cleaning and care of you intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion based on whey and stinging nettle with a low pH-value (3.8-4).

What can I do against increased discharge?

A slightly increased discharge is a common side effect of the treatment with Cerclage Pessaries. It may be avoided or reduced through thorough intimate hygiene.

Cerclage Pessary sizes

Cerclage Pessaries differ in their outer diameter (65 or 70 mm), the height of the curvature (17, 21, 25 or 30 mm) and the inner diameter (32 or 35 mm). A height of 17 or 21 mm is usually sufficient in the case of a singleton pregnancy. In the early weeks of pregnancy and/or in the case of descensus complaints as well as a multiple pregnancy a large model with a height of 25 or 30 mm is recommended.

Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

Do not use if you are oversensitive to silicone.
So far no allergic reactions known.

Infections of the vagina and the cervix are
contraindications for pessary treatment.

Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Maximum period of application with medical supervision: 2-3 cycles of 29 days maximum.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and/or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The application has to be documented by a gynaecologist.
- Have your gynaecologist check the pH-value and clean the pessary every or every other week.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

Additional products

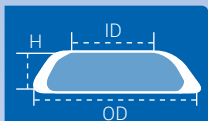
Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Cerclage Pessary

ID OD H

ID = Inner Diameter · OD = Outer Diameter · H = Height
All sizes in mm

Content:

- 1 pessary with instruction manual
- 1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

Usable until

Item number (MED...)

LOT

Attention

Date of construction

Read and follow the instructions for use

Producer

CE-mark with identification number of the notified body

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medesign I.C. GmbH